

751 / 751C OVERBED TABLES

Introduction

The Days Healthcare model no. 751 AND 751C are adjustable height overbed tables made from chromium plated and powder coated steel tubing and can also be used at a chair or wheelchair. The easy-clean laminated teak top has raised edges with open corners and can be tilted for reading.

The 751C model is fitted with castors which can be braked to avoid movement of the table when it is in the desired position.



751

- **CAUTION!** - The overbed tables are not designed to support heavy weight. Users must not lean on them or use them for support when getting up or sitting down.

Height Adjustment

The table height can be adjusted by slackening the two black hand knobs and adjusting the table top to the desired position then tightening the hand knobs.

Before use make sure that both hand knobs are securely tightened.



751C

Maintenance

The Days Healthcare overbed tables are designed for minimum maintenance.

To clean, wipe over with a soft cloth. Use general purpose neutral detergent and warm water. Do not use harsh abrasives.

To disinfect clean as above and wipe over with solution of one thousand parts per million available chlorine disinfectant (haztabs/precept), rinse and dry.

Specification

Table Size: 60 × 40 cm (24" × 15½")

Height Adjustment: 62 - 98 cm

Recommended Max. Load: 10 kg

Warranty Information

All Days Healthcare overbed tables are warranted for one year from the date of purchase against faulty workmanship or materials. Please contact your supplier/dealer should a fault occur.

The warranty does not extend to the consequential costs resulting from fault clearance, in particular freight and travel costs, loss of earnings, expenses, etc.

The manufacturer will not accept responsibility for any damage or injury caused by misuse or non-observance of the instructions set out above.