

TO DISPLAY THE TEST RESULTS:

After the test is completed, the Vacuum Indicator will automatically display the result from STEP 1. To display the results from STEP 2, 3 and 4, simply press the TEST-Button once for each STEP. If you continue pressing the button after the STEP 4 result has been displayed, the earlier results will be repeated (STEP 1,2,3,4,1,etc.). To exit the test program, set the Operating Knob to another position.

TEST STEP DISPLAYED ON THE BATTERY STATUS INDICATOR	TEST RESULT DISPLAYED ON THE VACUUM INDICATOR	ACTION IF TEST FAILED
STEP 1 - Occlusions	Test Passed: ≤ 100 mmHg.	Check possible occlusions (e.g. twisted tubing) and run the Device Test again.
STEP 2 - Vacuum Build-Up Efficacy - within 3 seconds	Test Passed: ≥ 300 mmHg.	Check Connectors, Tubes and Canister Lid for leakage or damage. Check exhaust outlet for occlusion and run the Device Test again
STEP 3 - Max. Achievable Vacuum - within 10 seconds	Test Passed: ≥ 500 mmHg.	Check Connectors, Tubes and Canister Lid for leakage or damage. Check exhaust outlet for occlusion and run the Device Test again
STEP 4 - Air Leakage	Test Passed: ≥ 450 mmHg.	Check Connectors, Tubes and Canister Lid for leakage or damage and run the Device Test again

Note: If the LSU does not pass one or more of the steps in this test after suggested actions are taken, the device might need to be returned for service.