

Antibody
Rapid Testing

MEDGUARD®



2019-nCoV Neutralization Antibody Test Kit



ACCURATE
High-purity antibody
with high accuracy



EFFICIENT
Test result available
in 15 minutes




CONVENIENT
Suitable for serum/
plasma/whole blood




Quick & easy testing In three simple steps


- 1 Add 20µl of whole blood sample/
10µl of serum or plasma sample
to well A.

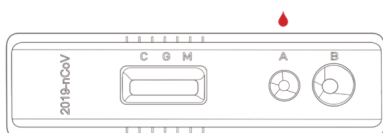

- 2 Add two drops (about 80µl) of sample
dilution to well B, and start timing.

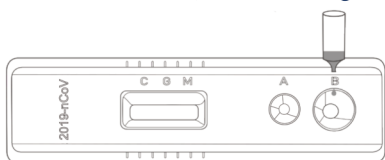

- 3 Interpret results at 15 minutes

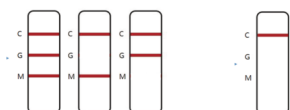

2019-nCoV Neutralization Antibody Test Kit Colloidal Gold Immunochromatography For Vaccine Effect Evaluation

This product is used to qualitatively detect 2019-nCoV neutralization antibodies in clinical samples (serum/plasma/whole blood).

Novel coronavirus (COVID 19) has rapidly swept across many countries since its outbreak, and it continues to spread all over the world. There is an immense pressure to detect, prevent & provide treatment. The long term solution against COVID 19 is to successfully develop a safe and effective vaccine. However, some neutralization antibodies are present in those infected with 2019 nCoV, and these antibodies can prevent the virus from reinfecting healthy cells by binding the spike protein with the virus surface.

Therefore, how to quickly and accurately detect the novel coronavirus (2019 nCoV) neutralization antibodies has become the top priority in accelerating the global anti epidemic process.

Clinical trials test results obtained with the Antibody Test Kit show that the diagnostic specificity, diagnostic sensitivity and the total coincidence rate are all greater than 90%.

Order Number	Description
SARS-COV-2-NEU	Sars-CoV-2 Rapid AntibodyTests (20)