

## SAMARIT Rollbord - WEIGHT LIMIT

### 1. Samarit Rollbord foldable

<i>Hightec, Professional, EcoLite, Blue Transfer Board</i>	150 kg	300 lbs
Operated by 2 carers and use of pull sheet	180 kg	360 lbs *

### 2. Samarit Rollbord NOT foldable

<i>Hightec, Professional, EcoLite, Blue Transfer Board</i>	150 kg	300 lbs
Operated by 2 carers and use of pull sheet	180 kg	360 lbs *

### 3. Samarit Rollbord ICU/Bariatric NOT foldable

<i>Hightec, reinforced innercore</i>	200 kg	400 lbs
Operated by 2 carers and use of pull sheet	230 kg	450 lbs *

\*With obese patients you must use a pull sheet in order to keep his “body fat” under control while transferring and pulling over the Rollbord. If the “body fat” cannot be controlled due to some unknown reason to you, we suggest using a lifter or sealing lifter for the security of the patient. With obese patients you must make sure that the support (stretcher/bed etc.) can hold the weight of the patient.

The above “weight limit” is based on different influences when a patient is transferred such as: Body weight, shape due to body fat, other health conditions, supporting stretcher as well as experience of staff in moving and handling.

The “weight limit” is a limit to improve the security of the patient. A well trained team than an unexperienced has a higher weight limit.

SAMARIT MEDICAL AG

January 2016

Medguard Professional Healthcare

Tel: +353 (0) 1 835 2411

Email: [sales@medguard.ie](mailto:sales@medguard.ie)

Fax: 01 969 5009

Website: [www.medguard.ie](http://www.medguard.ie)

Unit 6/7, Block 4, Ashbourne Business Park, Ashbourne, Co. Meath, A84 VK52